
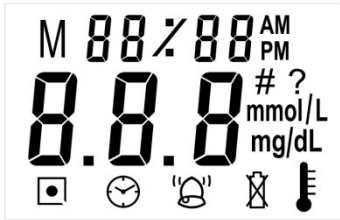


Taking a reading with the TD Glucose Patch


Press the **Function** button  on the side of the meter. The display will activate, showing all icons and segments activated for two seconds, then the time will appear with the current batch code, check batch code on package. Remove the patch from the foil pack and you are ready to start a test.




Power up




Batch Code Select

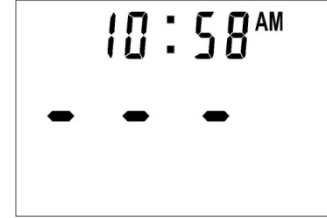


Ready Mode

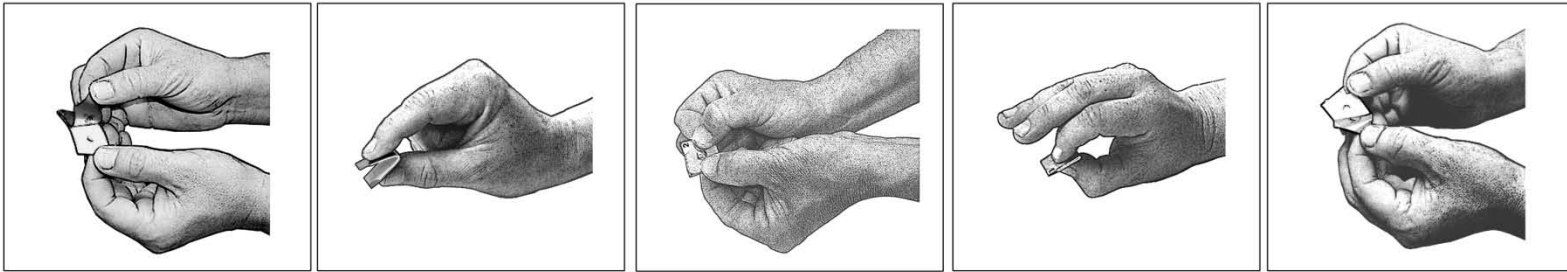

1. Remove the foil marked 1.
2. Fold the patch in half.
3. Squeeze all 4 sides of patch
4. Squeeze the center dimple
5. Remove the foil marked 2.

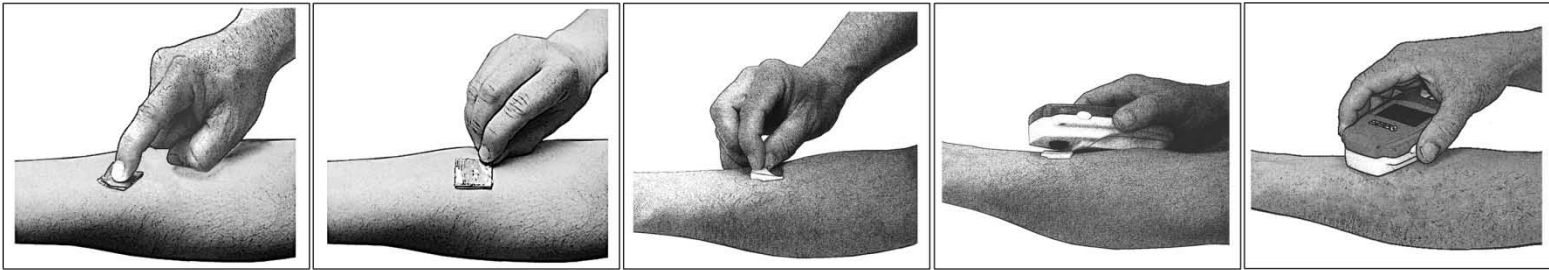

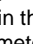6. Place the patch immediately on the inside of your forearm with the adhesive side down pressing the center of the patch to seal to your skin then press **OK**
7. After 5 minutes the meter will beep and the bell icon will blink, it is time to take a reading. Now from the undercut corner of the patch, as shown above, remove the foil side 3 exposing the center target hole in the patch, immediately press the **OK** button.
8. Immediately align the circular window of the sensor with the center hole in the patch, the red lights will help with alignment.
9. With the meter flat against the patch and with the window fully engaged in the center hole, hold the meter steady and press the  on the side of meter. within a few seconds the reading is displayed.

5 minute countdown



Time to take a reading



Note: Although the meter is designed to tolerate slight movement it is important to keep the meter square on the patch and to avoid excessive movement while the reading is taking place.

